

## Getting started:

Lock the push handles to the suitable height using the hand screws. They must be fastened tightly to avoid movement during use.

### NOTE!

- \* **Maximum permitted height:** The fourth bolt point should not be visible above the tubular frame.
- \* **The maximum allowed lateral adjustment angle of the handles is 0°.**
- \* **Always before use, check the operation of the brakes. Also make sure that the device is locked in the operating position and that all the wheels are evenly on the ground and that there are no distortions in the frame.**



## Parking brake:

To apply the parking brake, push both brake levers downwards. Handles will snap to a lock.

**Always lock the brakes when using the rollator as a seat.**



## Operating brake:

Pull upwards to use the brakes.



## Folding up:

Loosen the hand screws of the push handles, lower the handles to the lowest position and turn them over the seat. Tighten the screws. Lift the locking spring under the seat and tilt the rollator forward. The rear pipe is released and you can store and transport your rollator in a smaller space.

When lifting the product, for example, to a car, use the rear pipe. When lifting the walking aid, be careful and apply ergonomic lifting techniques.



**Make sure that your hands do not get caught between the locking spring and rear pipe or between the rear and front pipes!**

More comprehensive user manuals can be found here: [www.tukimet.fi/ohjekirjat](http://www.tukimet.fi/ohjekirjat)



**Watch the commissioning on YouTube:**

*Check regularly that all screws, nuts and other fasteners are properly tightened and that the brake is secure. Do not store the rollator outdoors. Keep the product clean of all dirt by wiping it with a damp cloth. Have a faulty product serviced at your own assistive device services facility!*